



Accessories

- Monitor with plugged-in cuff and batteries inserted
- Protective pouch with waistbelt
- Textile sleeve for upper arm
- Order form, patient record and instructions for use
- Red closure clip for the return

Return

After the recording has been completed, please return the monitor with the cuff plugged in, order form and patient records. Viollier will cover the shipping costs.

- By customer service – return to practice
- By DPD – drop off at a pickup parcelshop → dpgroup.com/ch/mydpd/parcel-shops
- By mail

Please note the following instructions for the return of the box:


- Remove label from window
- Slide the appropriate return label (in the case) into the window
→ **Viollier AG address is visible**
- Close case with red clip

Herz-Lungen-Praxis | Spalenring 147 | 4055 Basel | T +41 848 121 121 | viollier.ch
Monday – Friday 07:30 – 18:00 h

© by Viollier AG / 10.2023 / 15270



IN STRUCTION

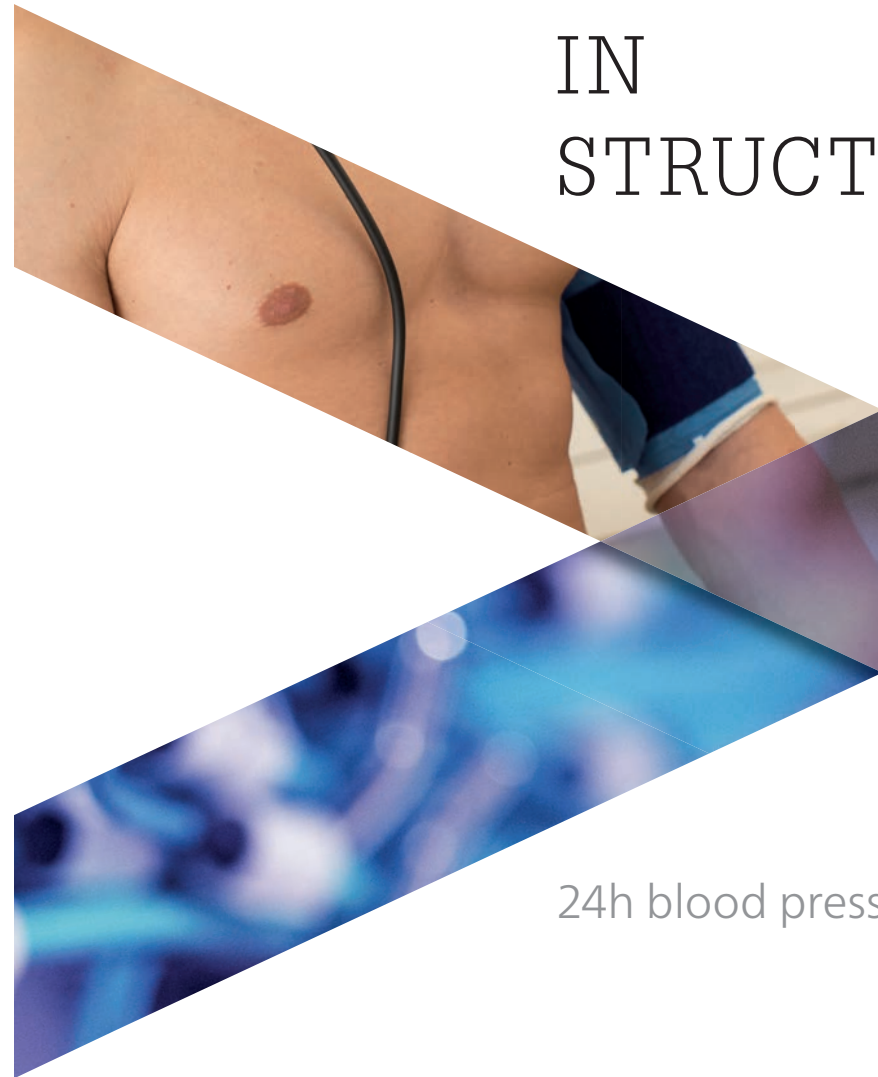

24h blood pressure

Important information

Start the monitor only when attached to the patient. Do not perform any trial measurements beforehand → monitor cannot be restarted.

- Request the suitable cuff size at Viollier: 20 – 24 cm = Children | 24 – 32 cm = Standard | 32 – 40 cm = X-Large | 38 – 50 cm = XX-Large
- The measurement intervals on the monitor are preprogrammed: every 15 minutes from 06:00 h, every 30 minutes from 10:00 h. If a faulty measurement is taken or if the blood pressure values are clearly raised, a further measurement is triggered after 2 minutes.
- To prevent faulty measurements during the blood pressure measurement, keep the arm still and extended downwards.
- Leave batteries in the monitor; do not remove.
- Mobile phones and strong electromagnetic sources can affect the quality of the recording.

Patient record

Please inform the patient about the diagnostic importance of filling out the patient record correctly and completely.

Please use a black ballpoint pen, not a pencil.

Note start and end of recording with date and time.

Enter date and time of symptoms, unusual activities, medicines, and sleeping period.


Identification and analysis

- Patient's personal details
- The date and exact start time of the recording, entered by you
- The monitor and cuff numbers entered by Viollier on the order form

| | | | |
|------------|-----|-----------------|-----|
| Geräte-Nr. | 022 | Manschetten-Nr. | 012 |
|------------|-----|-----------------|-----|


Please use the enclosed order form only, as this already has your address on it.

Attaching the monitor

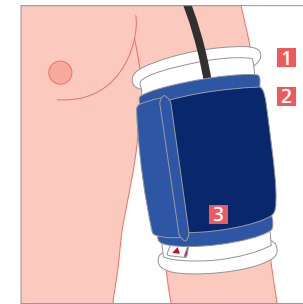

Mounting the monitor

- Pull the **textile sleeve 1** over left upper arm.
- Place the **cuff 2** over the textile sleeve and attach tightly.
- Position the **marking 3** on the inside (artery) of the upper arm.
- Place the **air tube 4** around the neck and feed it downwards on the right side of the body → **Air tube and cuff must not be glued.**

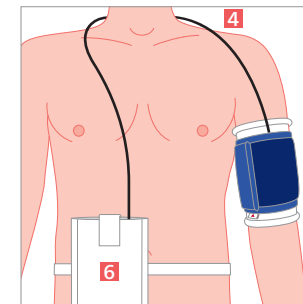

Starting the monitor

- Push the black **sliding switch 5** to position I → **Do not switch off again.**
- Wear the **monitor 6** in the protective pouch on the belt around the waist.
- Trigger the first measurement by pressing the **blue button 7**. If the measurement is performed correctly, the measurement value appears in the monitor display. If no result appears on the display but a number and the letter E, check that the cuff is correctly positioned, if necessary reposition it, and trigger a new measurement by pressing the blue button.
- After 24 hours switch off the monitor (position 0) **8**, dismount and return.

