


## Urine – chemical analyses: Spot urine

Spot urine tube without preservative (52) or spot urine tube with hydrochloric acid (HCl) (53)

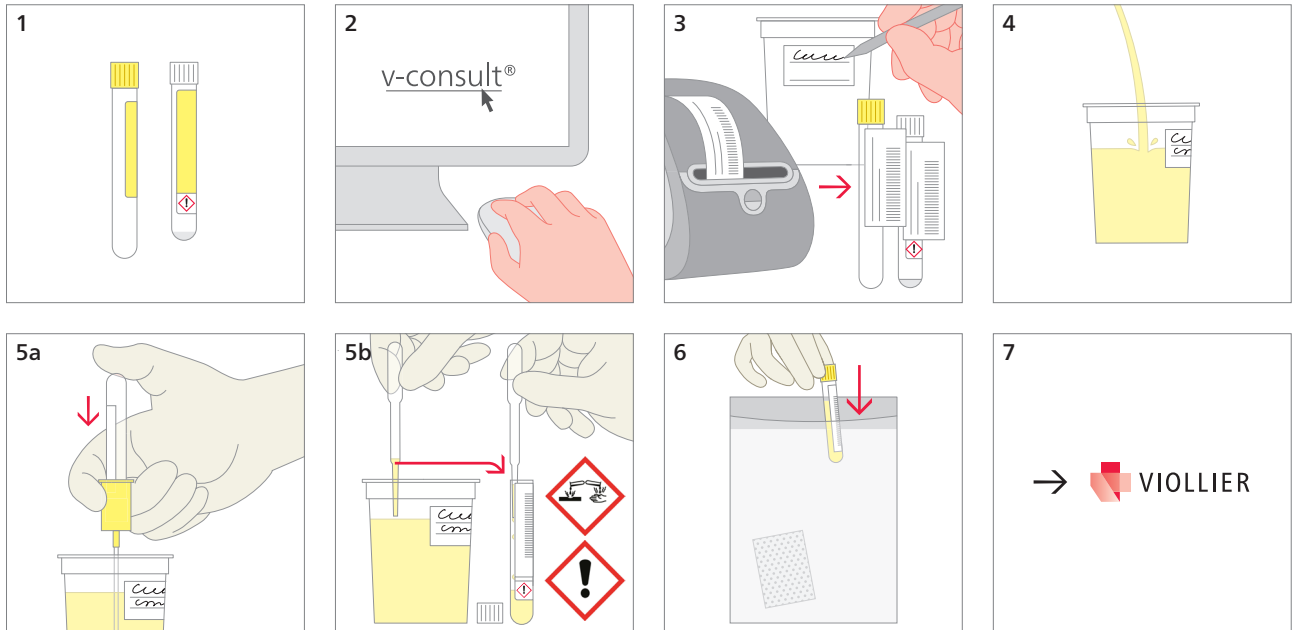

- 1** Select the correct spot urine tube either without preservative (52) or with hydrochloric acid (HCl) (53)

---

- 5a** Fill spot urine tube without preservative (52) completely with urine

---

- 5b** Fill spot urine tube with hydrochloric acid (HCl) (53) completely with urine  
 Caution: Hydrochloric acid is caustic and can cause skin and serious eye irritation!  
 Do not allow it to come into contact with mucous membranes. Keep away from children  
 → Tox Info Suisse emergency number 145 in case of contact with / corrosive injury from hydrochloric acid

---

- Number and type of required urine collection containers as per v-consult®

Shipping material:

Please note the last name, first name, and birthday of the patient